



Schild Line


FACE MASK

User Guide

Instructions for use:



1. Before applying the mask, wash your hands with soap and water or alcohol based hand sanitizers.
2. Place the white side of the mask so it completely covers the nose and mouth. Pull both sides of the ear straps and place one behind each ear.
3. The adjustable nose clip must sit at the bridge of your nose and the bottom of the mask must be fixed under your chin. This ensures that it covers your nose and mouth perfectly.
4. Take the mask off by pulling the bands. Do not touch the front blue side of the mask.
5. After removing the mask, immediately throw it in the disposal unit.

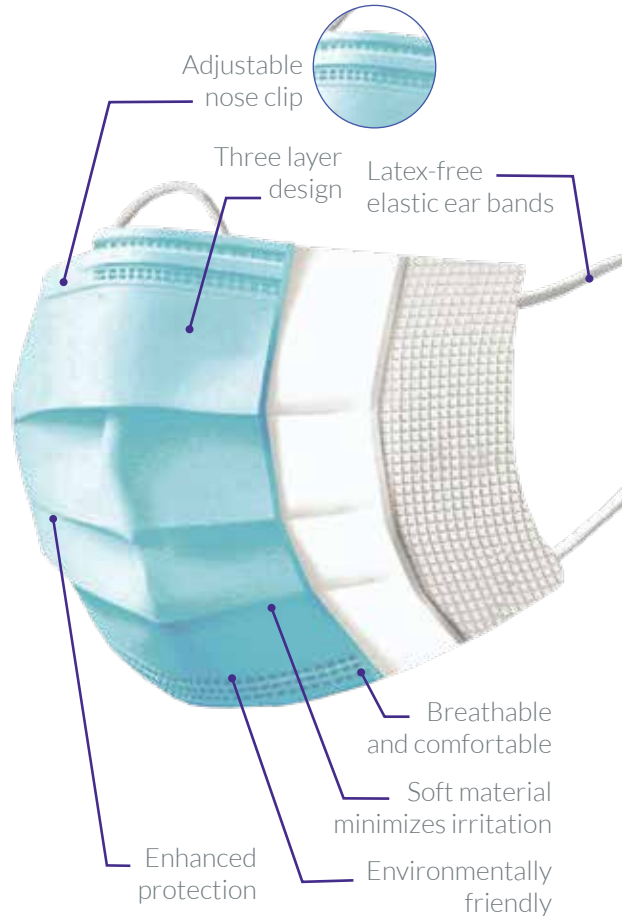

Precautions:

- This mask is not reusable. It must be disposed immediately after a day of use.
- If you experience an adverse reaction while wearing your mask please stop using it immediately.
- Check the validity date of the mask before using.
- Store in a dry area and keep it away from flammable objects.

Recommendations for use:

It is recommended that these masks be used in stores and shopping centres, on public transport, when meeting people and in work places where remote working is not possible.